

USE AND ADJUSTMENT OF FOLDING CANES

Your folding stick will be supplied in the folded position. It is made from several sections of aluminium tubing, linked together with an elastic bungee cord. When the folding stick is opened out to the extended position, this bungee cord ensures the sections of the stick remain held firmly together.

To unfold your walking stick:

Remove it carefully from its packaging. Holding all the sections together, remove the rubber band that is holding the stick in the folded position. Then carefully slot each tube into the next, starting at one end of the walking stick.

To adjust the height of your walking stick:

The top section of your walking stick comprises a smaller tube inserted into a larger tube, held in place by a brass security pin. On some models, the collar positioned on this section of the folding stick may need to be loosened. Then, depress the brass security pin and ease the inner tube downwards until the brass pin engages with the next hole. The brass pin should be fully engaged in whichever of the holes provided ensures the correct overall height for the folding cane. The collar should then be tightened again if necessary to ensure a firm fit.

To fold your walking stick:

To fold the walking stick, pull each section slightly away from the next and then fold the sections next to each other. Secure with a rubber band, or place inside a Classic Canes folding stick wallet.

If you have any further queries about the operation of your Classic Canes folding cane, please contact the retailer from which it was purchased.