

## TO REPLACE A METAL FERRULE

Place the new ferrule on the tip of the walking stick. Check the fit to ensure the ferrule is neither too large nor too small for the cane. With the ferrule in place, tap the end of the stick smartly on the ground once or twice to position the ferrule on the stick. Take care not to tap the stick too sharply as the varnish may be damaged if the ferrule is forced on too firmly.

Taking a small hammer and a fair sized nail, tap the nail into the soft metal of the ferrule just below the point where the ferrule and the stick meet (see photograph below). Repeat on the opposing side of the ferrule. These indentations will keep the ferrule on.



If you have any queries, please contact our sales office on Tel +44 (0)1460 75686.

Classic Canes Limited, Warren House, Hinton St. George, Crewkerne, Somerset TA17 8TQ, England  
Tel +44 (0)1460 75686 Fax +44 (0)1460 78222 info@classiccanes.co.uk www.classiccanes.co.uk

**Directors:** Charlotte Gillan, Diana C Porter

Registered in England and Wales No. 9583913. Registered office: Hendford Manor, Hendford, Yeovil, Somerset BA20 1UN

**Established 1982**

